



Changed  
Cartridge System

Changed  
Cartridge System

 **PRECISE IS PANASIL®**

**PRECISION IMPRESSION**


  
**KETTENBACHDENTAL**  
Simply intelligent

# PRECISION WITHOUT COMPROMISES

## Precision impression material (A-silicone)

In the **Panasil**<sup>®</sup> product family you find perfectly coordinated, innovative materials for all impression techniques and indications.

## DOUBLE-MIX IMPRESSIONS

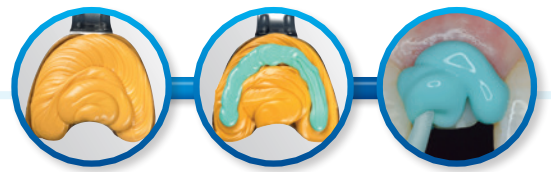

**Panasil**<sup>®</sup> tray Soft Heavy

**PLEASANTLY HEAVY-BODIED**



**Panasil**<sup>®</sup> initial contact Light




**Panasil**<sup>®</sup> contact two in one Light

**LOW VISCOSITY**

**INGENIOUSLY HYDROPHILIC**

**FAST SETTING TIME**

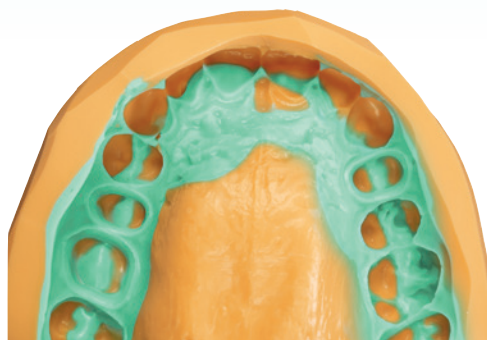

# FOR EVERY TECHNIQUE AND EVERY INDICATION

## TWO-STEP IMPRESSIONS



THE FAVOURITE



Changed Cartridge System

Panasil® binetics Putty Fast

REAL PUTTY MATERIAL,  
VERY EASY TO TRIM



Panasil® Putty Fast

GOOD VALUE FOR  
MONEY



Changed Cartridge System

Panasil® tray Fast Heavy

SHORT SETTING TIME



Panasil® initial contact X-Light




Panasil® contact plus X-Light

VERY LOW VISCOSITY

INGENUOUSLY HYDROPHILIC

FAST SETTING TIME

=

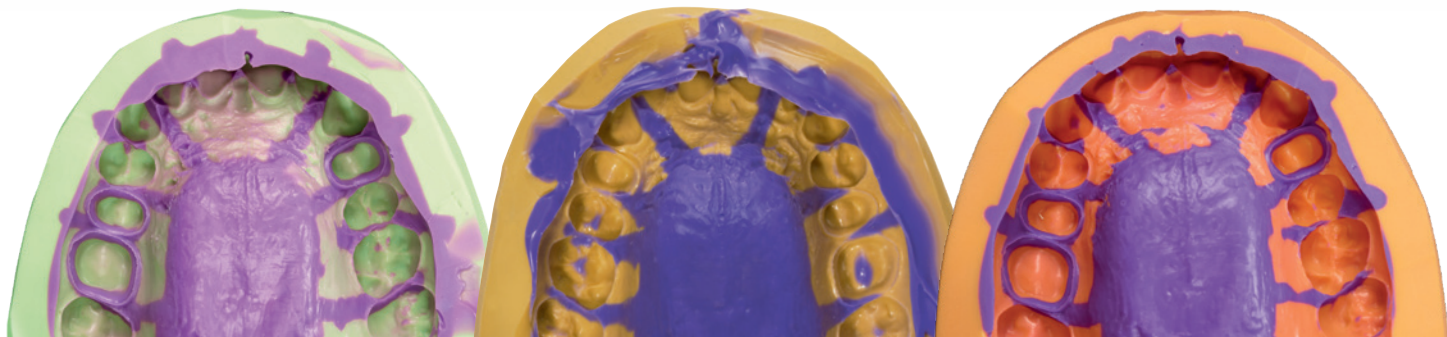



**KETTENBACHDENTAL**

Simply intelligent

## SANDWICH IMPRESSIONS



**Panasil®**  
binetics Putty Soft


**Panasil®** Putty Soft

**SMOOTH KNEADABLE, STABLE**



**Panasil®** initial contact Light

**LOW VISCOSITY,  
INGENIOUSLY HYDROPHILIC**

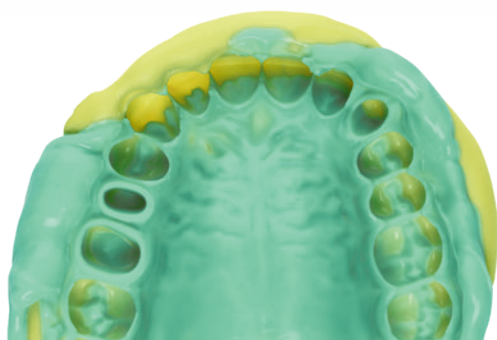

## MONOPHASE IMPRESSIONS



**Panasil®** monophase Medium

**HIGH FINAL HARDNESS**



**Panasil®** monophase Medium

**HIGH FINAL HARDNESS**




# OUTSTANDINGLY HYDROPHILIC FROM THE 1<sup>ST</sup> SECOND

## Contact angle measurement

(parameter for the hydrophilicity of a material)

The flatter the drop, the more pronounced the hydrophilicity of the material.

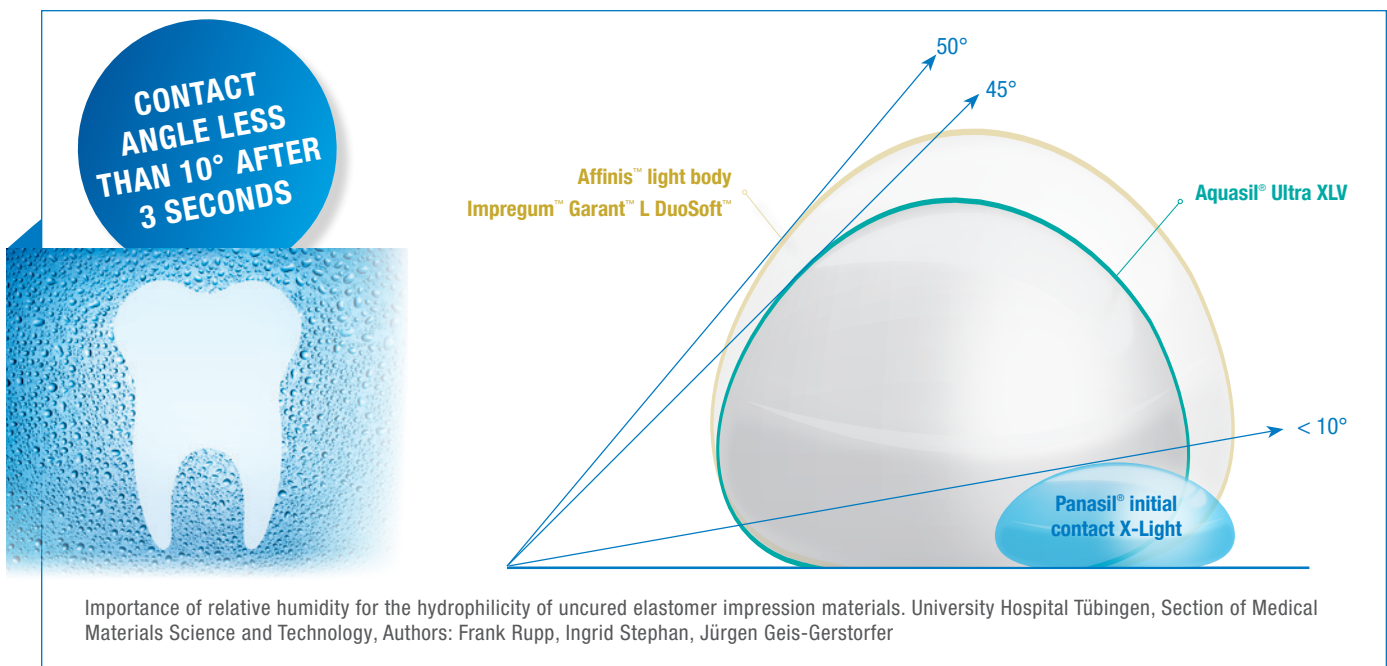

**Panasil® initial contact** shows extremely fast spreading of the water drop and reaches the lowest contact angle after 3 seconds compared to the competitive materials.

## EXCELLENT HYDROPHILICITY



- **Panasil® immediately overcomes any residual moisture** on direct contact with the tooth surface and ensures a precise result.



- **Panasil® ensures sharply defined preparation margins** even in **extreme situations**, e.g. in the case of difficult hemostasis.

# PANASIL® – THE RANGE

	Panasil® Putty	Panasil® Putty Soft	Panasil® Putty Fast
<b>PUTTY</b>	<b>Intro pack</b>	–	REF 11120
		200 mL base paste, 200 mL catalyst paste, 2 dispensing scoops	
	<b>Normal pack</b>	REF 11101	REF 11121
	450 mL base paste, 450 mL catalyst paste, 2 dispensing scoops		REF 11141
<b>Economy pack</b>	REF 11103	REF 11123	REF 11143
	4 x 450 mL base paste, 4 x 450 mL catalyst paste, 2 dispensing scoops		
	<b>Panasil® binetics Putty Soft</b>		<b>Panasil® binetics Putty Fast</b>
<b>Refill pack</b>	REF 14703		REF 14701
	2 x 380 mL cartridges		

	Panasil® tray Soft Heavy	Panasil® tray Fast Heavy	Panasil® tray Soft Heavy Fast
<b>HEAVY</b>	<b>Refill pack</b>	REF 14707	REF 14705
		2 x 380 mL cartridges	
<b>Normal pack</b>	REF 13541	REF 13551	REF 13561
	2 x 50 mL cartridges, 6 white mixing tips		

	Panasil® monophasic Medium
<b>MEDIUM</b>	<b>Refill pack</b>
	REF 14709
	2 x 380 mL cartridges
<b>Normal pack</b>	REF 13501
	2 x 50 mL cartridges, 6 white mixing tips

	Panasil® initial contact X-Light/ X-Light Fast	Panasil® initial contact Light/ Light Fast	Panasil® initial contact Regular
<b>LIGHT</b>	<b>Normal pack</b>	REF 13401 / REF 13461	REF 13411 / REF 13471
		2 x 50 mL cartridges, 8 grey mixing tips	2 x 50 mL cartridges, 6 white mixing tips
<b>Bonus pack</b>	REF 28300 / –	REF 28310 / –	–
	10 x 50 mL cartridges, 40 grey mixing tips		
	<b>Panasil® contact plus X-Light</b>		<b>Panasil® contact two in one Light</b>
<b>Normal pack</b>	REF 11892		REF 11781
	2 x 50 mL cartridges, 8 grey mixing tips		2 x 50 mL cartridges, 6 white mixing tips

All 380 mL and 50 mL cartridges are available in a changed cartridge system.

## NOTES

---



---



---



---



---

Kettenbach GmbH & Co. KG  
Im Heerfeld 7  
35713 Eschenburg · Germany  
Phone +49 (0)2774 7050  
info@kettenbach.com  
www.kettenbach-dental.com



142742\_0526



**KETTENBACHDENTAL**  
Simply intelligent